




- ✓ Each 0.90 mL cavity allows for customizable dosing with scored halves and quarters
- ✓ Easy-press buttons to eject full troches
- ✓ CLICK locking mechanism eliminates accidental opening
- ✓ Translucent tray allows for easy view of doses remaining
- ✓ Available in 4 colors to aid in medication differentiation

Patient Use Instructions



STEP 1:


Remove troche tray from protective cover.

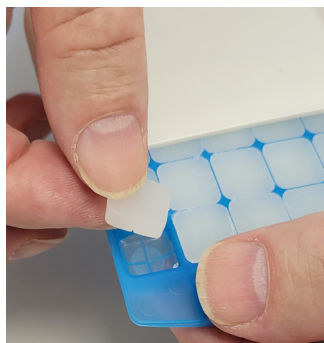

STEP 2:

Use Easy-press buttons to eject troche lozenge.

Dispense troche lozenge dosage as directed by your physician or pharmacist.

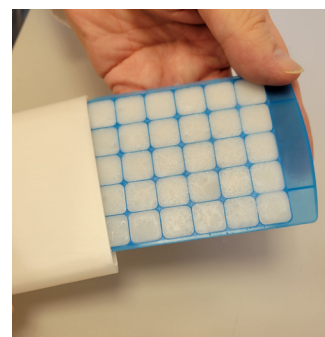

STEP 3:

Slide protective cover over tray to activate the CLICK locking mechanism.

Topi-CLICK® Troche-CLICK™ is Available in 4 Colors



Multiple Color Options make it easy for patients to distinguish between their medications.

Made of FDA grade materials | BPA and Phthalate free | Blocks UV.

Topi-CLICK® Troche-CLICK Patent Pending

Troche-CLICK Patient Use_102821

© 2021 TEAM Technologies, Inc. All Rights Reserved.

TO ORDER:

info.TopiCLICK@teamtech.com

| 877-870-8448

| www.Topi-CLICK.com