



*NEW Color
Now Available!*

**Topi-CLICK Perl®
proven
#1 in Accuracy¹**

¹Request the
American Journal of Internal Medicine
Perl® ARL Study Evaluation at
https://topi-click.com/PerlAccuracy_study/.



Topi-CLICK Perl® Vaginal Dosing Kit with Snap-on Vaginal Applicator




Topi-CLICK Perl® in RoseQuartz


Topi-CLICK Perl® in White

PERL® is a NEW way to dispense vaginal medications. The elegant Topi-CLICK Perl® Vaginal Dosing Kit combines the Topi-CLICK® 35 loader with the Perl® Snap-On Applicator™ resulting in a vaginal dosing method that is precise and designed for comfort. **(0.25 mL / CLICK)**

Snap... CLICK... Tilt... *GO!*™

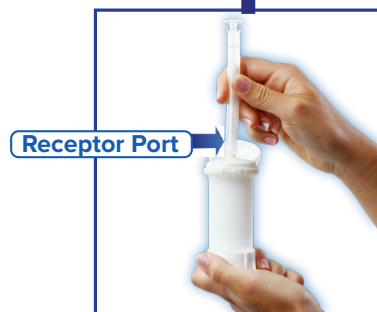

Snap On Applicator

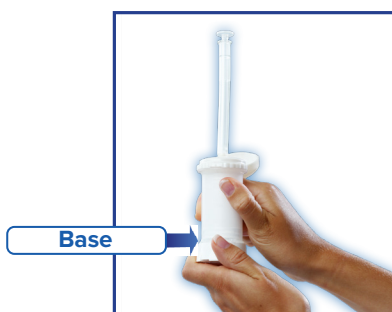

CLICK / Turn Base



Tilt Off Applicator

The Topi-CLICK Perl® dispenser is designed to be used exclusively with the Perl® Snap-on Applicator™.

© 2022 TEAM Technologies

Topi-CLICKPerl_010122

Topi-CLICKPerl_010122

TO ORDER:

info.TopiCLICK@teamtech.com

| 877-870-8448

| www.Topi-CLICK.com



Topi-CLICK Perl® Vaginal Dosing Kit

Snap... CLICK... Tilt... *GO!*™

A. Opening/Attaching the Perl® Snap-On Applicator™

1. Flip open the Perl shell cap. (Figure a.)
2. Snap on the applicator to the **receptor port** of the Perl dispenser. The applicator plunger should touch the **receptor port**. The **Topi-CLICK Perl dispenser is designed to be used exclusively with the Perl Snap-on Applicator™**. (Figure b.)


Flip Open



B. Loading Medication into the Perl® Snap-On Applicator™

1. Simply turn/CLICK the base of the Perl to load the metered dose into the applicator. (Figure c.)
2. Keep the Perl in vertical position when loading the applicator as shown. (Figure c.)
3. Do not push down on plunger.

Snap On

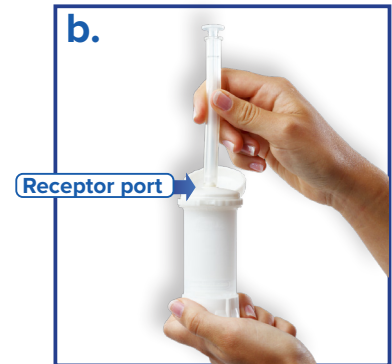


Note: Each turn/CLICK dispenses 0.25 mL of medication accurately and does not require fill lines on the applicator. (see table below)

Dosing Guide

1 CLICK = 0.25 mL	3 CLICKs = 0.75 mL
2 CLICKs = 0.50 mL	4 CLICKs = 1.00 mL

Turn / CLICK

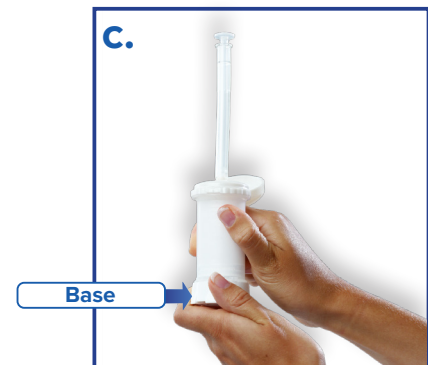

C. Removing Perl® Snap-On Applicator™


1. Slowly tilt off the Perl applicator as shown until the applicator releases from the receptor port. **Do not pull or snap off.** (Figure d.)
2. Use vaginal applicator to dose as prescribed.
 - **Applicator is dishwasher safe, but use caution to avoid residual medicine transfer when cleaning.**
 - **USP Class VI compliant**

Tilt off



D. Closing the Cap

1. Flip down shell cap tightly after each dose to prevent evaporation.

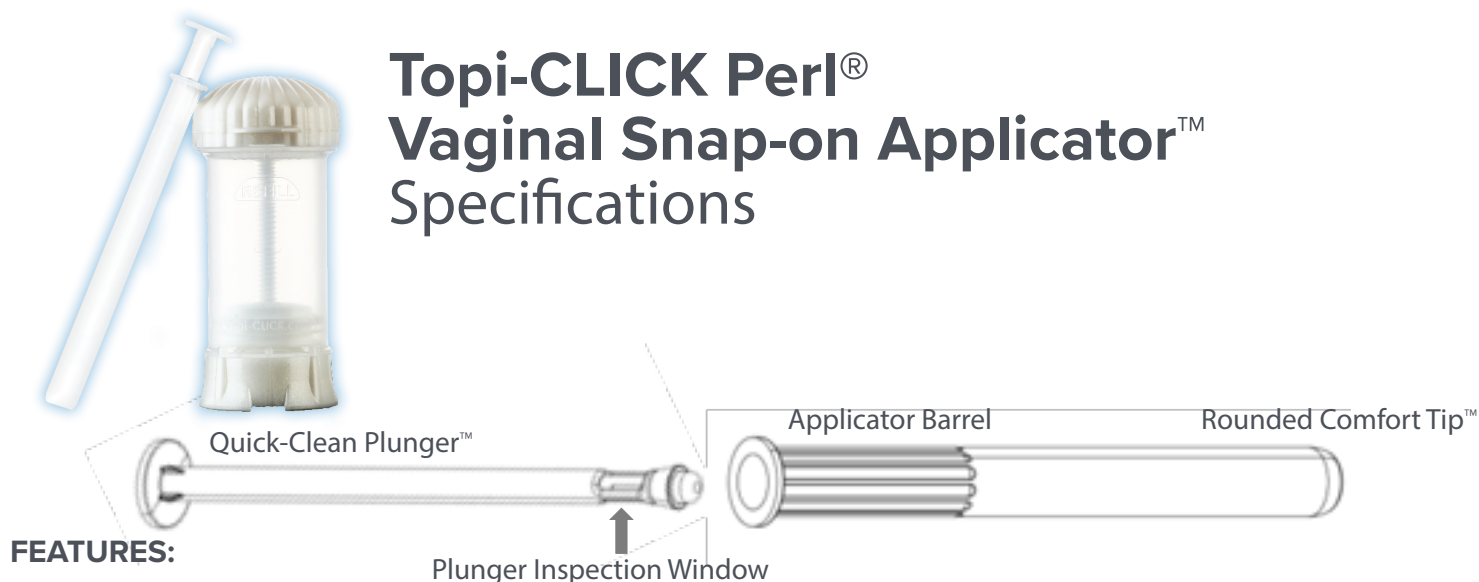



Topi-CLICK Perl® Snap-on Applicator™ Features



- Easy & Accurate Dosing compared to older style of threaded tip applicators for filling by tubes.
- Rounded Comfort Tip™ to help insertion be more pain-free and safer for patients.
- Snap-On Applicator Tip™ for fastest connecting and disconnecting during loading.
- Improved hygiene: Elimination of screw threads at tip allow for more thorough and quick cleaning.
- Quick-Clean Plunger™ design for easier cleaning.
- Maximum Applicator Capacity = 4.75 mL
- Minimized Residual as compared to older style of threaded tip applicators for filling by tubes.

Filled Amount	Threaded Ribbed	Threaded Smooth	Perl (Non-Threaded)
Filled to 1.0 mL: Residual =	0.10 mL	0.16 mL	0.04 mL
Filled to 4.0 mL: Residual =	0.25 mL	0.18 mL	0.16 mL

♦Applicator Barrel Material:

- Meets USP Class VI & ISO-10993
- BPA, PVC, PCB, and Phthalate Free 
- Reusable and does not degrade after 30 dishwasher cycles.
- Recycle code 4 

Quick-Clean Plunger™ Material:

- Meets USP Class VI & ISO-10993
- BPA, PVC, PCB, and Phthalate Free 
- Reusable and does not degrade after 30 dishwasher cycles.
- Recycle code 5 

*Volumes are approximate. As required for compounded prescription pharmaceuticals dispensed in volume dispensers; verify the weight to volume conversions thoroughly.

Topi-CLICK Perl® Vaginal Dosing Kit is the most accurate vaginal dosing method among those compared in an independent study titled *Single-Blinded Feminine Dosing Study* by Analytical Research Laboratories. Request study at https://topi-click.com/accuracy_study/.



Prescribe the
#1 in Accuracy
Topical Dosing Dispensers

Rx _____

Example

Compounded Medication

Generic Name of Active Ingredients(s) / Strength or Dose (i.e., mL or %)

Dosage Form: (i.e., Transdermal Cream, Transdermal Gel)

Quantity:

Directions for Use:

B1ES1 0.1/1.5
g: 2 cless apply to the
(topical)

Topi-CLICK Perl® Vaginal Dosing Kit Specifications

Topi-CLICK Perl® Vaginal Dosing Kit: 35 mL, 0.25 mL per CLICK

DESCRIPTION: Topi-CLICK Perl® (Precise / Easy / Reliable / Loading)
is a NEW way to dispense vaginal medications.

The elegant Topi-CLICK Perl® Vaginal Dosing Kit combines the Topi-CLICK® 35 loader with the Perl® Snap-On Applicator™ resulting in a vaginal dosing method that is precise and designed for comfort.

The Topi-CLICK® Perl® can be used to dispense approximately 5 mL to 35 mL of a topical preparation depending on the volume to weight ratio of base and formula used.




FEATURES:

- Each CLICK or turn of the base moves the plunger approximately 0.25 mL of volume space, thereby dispensing 0.25 mL of cream on average. The most common number of CLICKs used for one dose is 2 CLICKs or 0.50 mL volume space.
- Audible, visual and tactile CLICK registrations for user recognition.
- Lower cost with simplified labeling – labels as fast and easily as a prescription bottle.
- Built-in applicator to prevent medication transference to unintended areas.
- “Refill” reminder badge helps patients to reorder in a timely manner.
- Designed to resist accidental discharge in purse, pocket or luggage.

TECHNICAL SPECIFICATIONS: Topi-CLICK Perl® Vaginal Dosing Kit

Dispenser Volume:	Up to 37 mL
Delivery Volume:	Up to 34 mL
Dispense per CLICK:	0.25 mL
Approximate Residual:	2.5 mL
Approximate Priming:	0.50 mL or 2 CLICKs
Doses Provided:	When filling with 35 mL, the Topi-CLICK® will provide approximately 32 mL or 64 doses (at 2 CLICKs per dose or 128 CLICKs total.)

- Registered FDA device, Class 1 Dispenser.
- Topi-CLICK’s FDA Registration # is 3007142798 and the Device Listing Number is D003073.
- Materials compliant with FDA 21.177 code, which is the requirement for United States Pharmacopeia (USP <661>).
- Made of FDA grade materials | BPA and Phthalate free | UV block.
- Recycling Code is #5 . Visit <http://doselogix.com/recycling-your-topi-click> for more information on recycling Topi-CLICK®.

*Volumes are approximate. As required for compounded prescription pharmaceuticals dispensed in volume dispensers, verify the weight to volume conversions thoroughly. See dispensing instructions at <http://doselogix.com/downloads-how-it-works>. One time use only.

Topi-CLICK Perl® Vaginal Dosing Kit is the most accurate vaginal dosing method among those compared in an independent study titled *Single-Blinded Feminine Dosing Study* by Analytical Research Laboratories. Request study at https://topi-click.com/accuracy_study/.

Topi-CLICK® Filling Instructions

Read all information, specifications and instructions carefully before filling and dispensing Topi-CLICKs.



STEP 1:

Fill the Topi-CLICK® dispenser with the desired amount of topical formulations using a Topi-CLICK® Filling Station, a spatula, EMP jar, or nozzle.



STEP 2:

If needed while filling, stop periodically to tap the Topi-CLICK® on the palm of your hand, counter or Tap-n-CLICK™ to help settle the topical.



STEP 3:

Put the built-in applicator (cap with a hole) on the top of the Topi-CLICK® container by aligning the inside key tabs with the slots on the top of the container tube and snap on.



STEP 4:

Prime the Topi-CLICK® by turning the base until topical is dispensed and wipe off topical.

Be sure to leave the base in the registered CLICK position by aligning the base column with the tube node: See Step 3 of the Patient Use Instructions.



Registered Position



STEP 5:

Snap on the protective outer cap. (If the cap doesn't fit tight, it's an indication that the built-in applicator in #3 is not "snapped" down all the way.)

Note: Be sure to align ports when using the 3Port™ cap and applicator.



STEP 6:

Apply center of label to the side of the Topi-CLICK® container which is opposite the "REFILL" and graduation marks.

For more information on dispensing amounts per CLICK see the Product Specification's at Topi-CLICK.com. See Patient User Instructions for information on how to apply topical medications using Topi-CLICK. Verify formulas and weight per volume as required for compounded prescription pharmaceuticals.