

# Topi-CLICK<sup>®</sup>

Accurate & easy dosing for pets

Accurate & easy dosing  
for pets



CLICK... Lick...  
goi<sup>™</sup>



Choose Topi-CLICK<sup>®</sup> for The Best User Experience<sup>™</sup>



Automated metered-dosing



Built-in dosing applicator



Audible, Tactile and Visual CLICKs indicate measure of dose (no need to manually mark dose)



Pets learn to associate the CLICK as positive conditioning so there is no struggle when dosing



Built-in REFILL Badge helps pet owners reorder medication on-time

## Lickable Suggestions

Amlodipine  
Atenolol  
Ciprofloxacin  
Doxycycline  
Fluconazole  
Fluoxetine  
Gabapentin  
Methimazole  
Mirtazepam  
Metronidazole  
Phenobarbital  
Piroxicam  
Potassium Bromide  
Ursodiol



Follow instructions on provider's label before using the Topi-CLICK<sup>®</sup>. Consult with literature or a prescriber veterinarian on the most appropriate route of delivery and dosing before dispensing to patients.

Request FREE Topi-CLICK<sup>®</sup> 35 samples for patient trials and also get a copy of the accuracy study at <https://topi-click.com/samples/>.

© 2022 TEAM Technologies

Topi-CLICK35VetUse\_010122

TO ORDER:

[info.TopiCLICK@teamtech.com](mailto:info.TopiCLICK@teamtech.com)

| 877-870-8448


| [www.Topi-CLICK.com](http://www.Topi-CLICK.com)

# CLICK... Lick... **Topi-CLICK®**

Accurate & easy dosing for pets



## Topi-CLICK® Pet Use Instructions

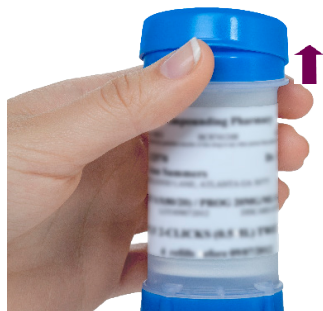

### STEP 1:

Lift off the cap of the Topi-CLICK® container by placing your thumb under the tab on the protective outer cap and pressing upward.



### STEP 2:

Turn the base to dispense the number of CLICKs specified in the instructions on your prescription label.



### STEP 3:

Watch the visual indicators when rotating base. Be sure to leave the base in the registered CLICK position by aligning the base column with the tube node after each use.

**Each turn/CLICK of the base dispenses 0.25 mL of medication accurately.**



### STEP 4:

Hold Topi-CLICK® while your pet licks the applicator pad to get an accurate tasty dose of medication.



### STEP 5:

Replace cap after each dose to prevent evaporation. Be sure cap snaps on tightly.

**Note** Be sure to align ports when using the Topi-CLICK® 35 3Port™ cap and applicator.



### STEP 6:

The bottom of the plunger is the indicating line; when it reaches the embossed line below the word "Refill" approximately 32 CLICKs are left. Approximately 12 CLICKs remain at the line above the word "Refill".

*Follow instructions on provider's label before using the Topi-CLICK®. Consult with literature or a prescriber veterinarian on the most appropriate route of delivery and dosing before dispensing to patients.*

**\*Topi-CLICK® 35 is the most accurate topical dosing dispenser among those compared in an independent study titled *Topical Metered Dosing Dispenser Performance Evaluation* by Analytical Research Laboratories. Request study at [https://topi-click.com/accuracy\\_study/](https://topi-click.com/accuracy_study/).**

# Topi-CLICK MICRO<sup>®</sup>

Revolutionary way to accurately & easily micro-dose topical medications for humans & pets



0.05 mL  
Per  
CLICK



NEW Sizes & Colors Available  
Topi-CLICK Micro<sup>®</sup> 9 mL &  
Topi-CLICK Micro<sup>®</sup> 5 mL

Easy Screw Cap



Choice of three applicators



Standard Luer Lock Connection



REFILL Indicator



## The Topi-CLICK Micro<sup>®</sup> Difference

### Accurate & Consistent Dose<sup>™</sup>

- Topi-CLICK Micro<sup>®</sup> is the most accurate among those in a patient simulation test.
- Eliminates user-variability
- Consistent automated dosing throughout treatment

### Automated Metered Dosing

- Audible, Tactile and Visual CLICKs indicate measure of dose at each point of the unique triad-shaped base (One CLICK = 0.05 mL)

### Targeted Topical Therapy

- Innovative triad-shaped dispenser allows for precision handling during application
- Applicator reduces the risk of hand transference to others
- Offers applicator selections for customized dosing

Select one of 3 applicators



Universal  
Rounded  
Applicator<sup>™</sup>



Soft  
Angled  
Applicator<sup>™</sup>



Pin Point  
Applicator<sup>™</sup>

### Viscosity Assurance<sup>™</sup>

- Accuracy NOT affected by varying formulations
- Accuracy NOT affected by varying bases

## Topi-CLICK Micro<sup>®</sup> 9 mL

Dispense/CLICK: 0.05 mL

Dispenser Volume: Up to 9.75 mL

Total Delivery Volume: Up to 9.45 mL

Approx. Residual: 0.15 mL

Approx. Priming: 0.15 or 3 CLICKs

## Topi-CLICK Micro<sup>®</sup> 5 mL

Dispense/CLICK: 0.05 mL


Dispenser Volume: Up to 5.5 mL

Total Delivery Volume: Up to 5.35 mL

Approx. Residual: 0.15 mL

Approx. Priming: 0.15 or 3 CLICKs

Innovative  
TRIAD Shape  
Allows Precision  
Application  
Control

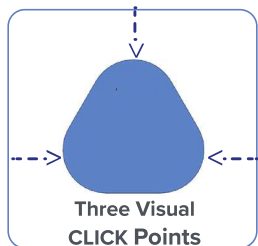

Each turn/CLICK of  
Base dispenses 0.05 mL  
of topical

1 CLICK = 0.05 mL  
2 CLICKs = 0.10 mL  
3 CLICKs = 0.15 mL



Base

Made of FDA grade materials | Phthalate free | blocks UV and is Recyclable ♻️.

© 2022 TEAM Technologies

Topi-CLICK\_MICRO\_010122

TO ORDER:

info.TopiCLICK@teamtech.com

| 877-870-8448

| www.Topi-CLICK.com

Topi-CLICK | TEAM Technologies | 300 Parkbrooke Place, Suite 140, Woodstock, GA 30189

TT HELPING THOSE WHO HELP OTHERS


# Topi-CLICK MICRO® User Instructions



- ✓ Automated, metered topical dosing provides an accurate and consistent dose every time
- ✓ Audible, Tactile and Visual CLICKs indicate measure of dose (no need to manually mark dose)
- ✓ Innovative triad-shape allows for precision handling
- ✓ Built-in REFILL Badge helps patients reorder medication on-time



**STEP 1:**  
Unscrew the cap of the Topi-CLICK Micro® and set aside for recapping after use.

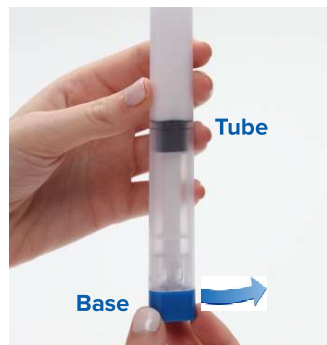

**STEP 2:**  
Holding the Micro® in an upright position, turn/CLICK the **Base** the number of CLICKs or mLs specified on the prescription label. Use the Audible, Tactile and Visual CLICK points of the triad-shaped **Base** as dosing indicators. Be sure to align **Base** with the **Triad Tube** to ensure dose is fully dispensed.



**STEP 3:**  
Apply topical medication with built-in applicator as directed by physician or pharmacist. **Continue rubbing until preparation is completely absorbed into the skin.**  
**IMPORTANT:** The built-in applicator lessens the risk of transference to unintended areas. No need to use your fingertips to apply medication.



**STEP 4:**  
Screw on cap after each dose to prevent evaporation. Be sure cap is secured completely.

Innovative  
TRIAD Shape  
Allows Precision  
Application  
Control

Each turn/Click of the base  
dispenses 0.05 mL of  
medication accurately.

## REFILL Badge™



REFILL  
Line

Plunger

**The Refill Badge™ reminder system helps patients reorder in a timely manner.**

When the **TOP** black line of the **Plunger** reaches the embossed line below the word "REFILL" approximately 32 CLICKs are left. Approximately 12 CLICKs remain at the line above the word "REFILL".

*Note: Graduated Marks are for pharmacy filling purposes only. Follow instructions on prescription label before use. Topi-CLICK Micro® is not re-fillable.*

Made of FDA grade materials | Phthalate free | blocks UV and is Recyclable ♻️.

# **Laboratory Test Results**

Product performed per design specifications. Device is both highly accurate and precise, demonstrating a tight variance and an average dose being 98.7% of the rated dose. Product performance variance was reduced for two CLICKs versus one CLICK.

Qualifies inspection and is passed.

## Accurate Micro-dosing Designed for Patients and Pets

### Topi-CLICK Micro® Lifetime Dosing Accuracy

(1 CLICK or 0.05 mL)

(2 CLICKS or 0.10 mL)

Average Dose

0.049 mL

0.099 mL

Standard Deviation

0.003 mL

0.004 mL

Coefficient of Variation

5.7%

3.8%

Average Dose to Rated Volume

98.7%

98.7%

#### Topi-CLICK Micro® 9 mL

Dispense/CLICK: 0.05 mL

Dispenser Volume: Up to 9.75 mL

Total Delivery Volume: Up to 9.45 mL

Approx. Residual: 0.15 mL

Approx. Priming: 0.15 or 3 CLICKs

#### Topi-CLICK Micro® 5 mL

Dispense/CLICK: 0.05 mL

Dispenser Volume: Up to 5.5 mL

Total Delivery Volume: Up to 5.35 mL

Approx. Residual: 0.15 mL

Approx. Priming: 0.15 or 3 CLICKs

\*Volumes are approximate. As required for compounded prescription pharmaceuticals dispensed in volume dispensers; verify the weight to volume conversions thoroughly.

Made of FDA grade materials | Phthalate free | blocks UV and is Recyclable ♻️