

# Topi-CLICK<sup>®</sup>

Accurate & easy dosing for pets

Accurate & easy dosing  
for pets



CLICK... Lick...  
goi<sup>™</sup>



Choose Topi-CLICK<sup>®</sup> for The Best User Experience<sup>™</sup>



Automated metered-dosing



Built-in dosing applicator



Audible, Tactile and Visual CLICKs indicate measure of dose (no need to manually mark dose)



Pets learn to associate the CLICK as positive conditioning so there is no struggle when dosing



Built-in REFILL Badge helps pet owners reorder medication on-time

## Lickable Suggestions

Amlodipine  
Atenolol  
Ciprofloxacin  
Doxycycline  
Fluconazole  
Fluoxetine  
Gabapentin  
Methimazole  
Mirtazepam  
Metronidazole  
Phenobarbital  
Piroxicam  
Potassium Bromide  
Ursodiol



Follow instructions on provider's label before using the Topi-CLICK<sup>®</sup>. Consult with literature or a prescriber veterinarian on the most appropriate route of delivery and dosing before dispensing to patients.



Request FREE Topi-CLICK<sup>®</sup> 35 samples for patient trials and also get a copy of the accuracy study at <https://topi-click.com/samples/>.

© 2022 TEAM Technologies

Topi-CLICK35VetUse\_010122

TO ORDER:

[info.TopiCLICK@teamtech.com](mailto:info.TopiCLICK@teamtech.com)

| 877-870-8448

|


[www.Topi-CLICK.com](http://www.Topi-CLICK.com)

# CLICK... Lick... **Topi-CLICK®**

Accurate & easy dosing for pets



## Topi-CLICK® Pet Use Instructions

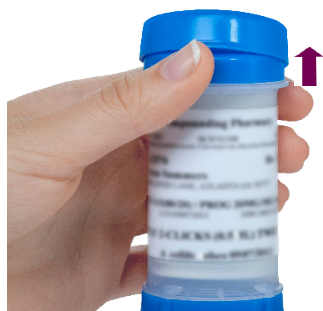

### STEP 1:

Lift off the cap of the Topi-CLICK® container by placing your thumb under the tab on the protective outer cap and pressing upward.



### STEP 2:

Turn the base to dispense the number of CLICKs specified in the instructions on your prescription label.



### STEP 3:

Watch the visual indicators when rotating base. Be sure to leave the base in the registered CLICK position by aligning the base column with the tube node after each use.

**Each turn/CLICK of the base dispenses 0.25 mL of medication accurately.**



### STEP 4:

Hold Topi-CLICK® while your pet licks the applicator pad to get an accurate tasty dose of medication.



### STEP 5:

Replace cap after each dose to prevent evaporation. Be sure cap snaps on tightly.

**Note** Be sure to align ports when using the Topi-CLICK® 35 3Port™ cap and applicator.



### STEP 6:

The bottom of the plunger is the indicating line; when it reaches the embossed line below the word "Refill" approximately 32 CLICKs are left. Approximately 12 CLICKs remain at the line above the word "Refill".

*Follow instructions on provider's label before using the Topi-CLICK®. Consult with literature or a prescriber veterinarian on the most appropriate route of delivery and dosing before dispensing to patients.*

**\*Topi-CLICK® 35 is the most accurate topical dosing dispenser among those compared in an independent study titled *Topical Metered Dosing Dispenser Performance Evaluation* by Analytical Research Laboratories. Request study at [https://topi-click.com/accuracy\\_study/](https://topi-click.com/accuracy_study/).**